FertiloScope™

INSTRUCTIONS FOR USE

INTENDED USE:
FERTILOSCOPE FOR HYSTEROSCOPY AND
TRANSVAGINAL HYDROLAPAROSCOPY

SINGLE USE ONLY: RE-USE WILL COMPROMISE
STERILITY AND INTEGRITY OF
BALLOONS & CATHETERS

ONLY TO BE USED BY A QUALIFIED SURGEON WHO
HAS RECEIVED APPROPRIATE INSTRUCTION

Contents:

Language	Page
English	2
Français	3
Dansk	4
Deutsch	5
Español	6
Italiano	7
Nederlands	8
Norsk	9
Português	10
Suomi	11
Svenska	12

Reorder number
FTO1.40

Each kit comprises:

1 x FTO 1.40 cannula, with its pointed trocar, devised for introduction in the pouch of Douglas in order to perform a Transvaginal hydro-pelviscopy.

For use with (re-order separately):

FH 1.06 cannula, devised for introduction in uterus in order to perform tubal patency test and hysteroscopy.

Support and Complaints
support@fertility-focus.com

Ordering
orders@fertility-focus.com

Web
www.fertility-focus.com

Instructions for Use Version
F002-18JAN16-V2.0-EN,FR,DA,DE,ES,IT,NO,PT,FI,SW

INTENDED USE: FERTILOSCOPE FOR HYSTEROSCOPY AND TRANSVAGINAL HYDROLAPAROSCOPY

With the following procedures: Transvaginal hydropelviscopy with a tubal patency test; Salpingoscopy and Microsalpingoscopy; Hysteroscopy. Performed in this order under local or neuroleptic sedation

Before starting Fertiloscopy, the following must be carried out on a patient already in the gynecological position with a speculum inserted:

- Vaginal asepsia with polyvinyl-pyrrolidone (Betadine®)
- Exposition of the cervix using a tenaculum at the 8 o'clock position
- Local anesthesia using 5ml of Xylocaine injected into each utero-sacral ligament or by neuroleptic sedation.

PRECAUTIONS

1. Check that the packaging is unbroken
2. Check that neither of the catheters have been damaged during the transport
3. Check that the balloon on the FTO 1.40 is functioning correctly before insertion
4. Check the integrity of the balloon on FTO 1.40 after withdrawal of the cannula. In the highly unlikely event that part of the balloon is missing after withdrawal, a laparoscopy audit would be required.
5. Do not use instruments with dimensions other than those indicated below
6. Only use instruments with these products that have individual electrical and radiation insulation and with an operating temperature of less than 50°C.

RISKS INHERENT IN THE TECHNIQUE

1. With FH 1.06 (supplied separately) or equivalent HSG catheter, as with the insertion of any device into the uterus, there is a risk, however slight, of perforation. If this should occur, rigorous monitoring should be maintained (risk of bleeding and/or infection).
2. With FTO 1.40,
 - a. The introduction of the cannula into the pouch of Douglas will always present a risk of sub-peritoneal rectal perforation. A gynecological and ultrasonographic examination of the pelvis should always be performed in order to verify the vacuity of the pouch of Douglas.
 - b. If, in spite of the above precautions, a perforation should occur, treatment with antibiotics and several days of observation should be sufficient.
 - c. Several cases of a sub-peritoneal trajectory of the cannula have been reported which subsequently require the removal of the cannula. For this reason, the balloon device should only be inflated once the correct positioning of the cannula in the pouch of Douglas has been verified through the scope.
 - d. Particular care must be taken when performing Hydropelviscopy in a potentially infectious environment (salpingitis by Chlamydia in particular). A relevant antibiotic treatment should be used in these cases.

A single-use sterile product - EO Sterilized. Re-use will compromise sterility and integrity of balloons and catheters. **ONLY TO BE USED BY A QUALIFIED SURGEON WHO HAS RECEIVED APPROPRIATE INSTRUCTION**

INSTRUCTIONS FOR USE

FertiloScope™ FTO 1.40

Cannula, with its pointed trocar, devised for introduction in the pouch of Douglas in order to perform a Transvaginal hydro-pelviscopy.

Prior to the use of the cannula, the following steps should be performed;

- Introduce FH 1.06 or equivalent standard HSG catheter and attach to patient's leg following the instructions supplied with that catheter
 - Introduce a standard 15cm long Veres needle 1cm below the cervix. If the uterus in an axial or in a mobile retroverted position, the plane of introduction is strictly mid-line and parallel to the axis of the posterior vaginal wall (when the speculum is opened widely). The more anteverted the uterus, the closer the plane of the needle introduction should be to the horizontal.
 - Instil 200ml of saline solution pre-heated to 37°C. The Veres needle is then removed.
1. Open package, place catheter on a sterile field
 2. Remove the protective safety clip located at the upper part of the trocar.
 3. Take the cannula and unscrew the central light blue cap to allow the pointed trocar tip to protrude from the end of the cannula.
 4. Connect the syringe provided to the green valve and inflate the balloon to check that it is functioning correctly.
 5. The non-return valve is automatically closed when the syringe is removed.
 6. Insert the cannula in the pouch of Douglas using the same perforation created by the Veres needle at the beginning of the procedure.
 7. Remove the trocar
 8. Introduce the scope, the device allows the use of any scope up to 4.1mm outer diameter. [Storz 2.9mm with sheath or 4mm without, or equivalent]. During the hydropelviscopy, irrigation can be performed through the sheath of the scope or through the working channel of the cannula (red connector). Always use saline iso-osmotic solution pre-heated to 37°C.
 9. Inflate the balloon with no more than 5ml of air
 10. Hydropelviscopy can now be performed
 11. Salpingoscopy and micro-salpingoscopy can be performed by introducing grasping forceps in the 5 French working channel in order to stabilize the fimbria prior to introducing the scope.
 12. At the end of the procedure, deflate the balloon by pressing the syringe firmly into the open non-return valve and withdrawing the syringe plunger
 13. Allow the saline to freely evacuate the abdominal cavity as necessary, through the cannula.
 14. Remove the cannula

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